GIBAUD

LOMBOGIB®
BB

6750

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EN | Instructions for Use

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If you have any doubts regarding the applications and use of this medical device, please ask your doctor and/or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

INDICATIONS

For lower back pain that may benefit from sacroiliac and suprapubic support, particularly in the following cases:

- Pregnancy-related back pain.

PERFORMANCE

The device provides support.
The device eases lower back pain.

CONTRAINDICATIONS

No known contraindications.

WARNINGS • PRECAUTIONS OF USE

The device must not be used directly on damaged skin.

This product contains Natural Rubber Latex (in the elasticated straps) which may cause allergic reactions.

The device must never compress the abdomen and must not be worn for prolonged periods without medical follow-up.

The device must only be used by a single patient. Do not use for multiple patients. If you experience any problems while using this device, remove it and contact your doctor.

Any serious incident involving the device must be reported by the healthcare professional and/or the patient to the manufacturer and the competent authority for their country.

FEATURES OF THE DEVICE

(illustration 1)

The Lombogib® BB is an elasticated lumbar support belt with four V-shaped dorsal ribs. The device includes:

- **(1A)** an elasticated lumbar belt 26 cm high at the back,
- **(1B)** two abdominal supports (small or large based on the waist size chart),
- **(1C)** hook and loop fasteners with guards to be removed before use and put back on before washing,
- **(1D)** a soft, adjustable, and removable back pad.

APPLICATION (illustration 2)

A fitting is required to ensure a good fit and a good understanding of how to

put the device on. A qualified healthcare professional must fit the device on the patient the first time. Subsequently, the patient herself or a family member or friend may put the product on her in the same way.

During the fitting, the professional must:

- **(2A)** Measure the waist circumference 2 cm above the iliac crest to choose the right abdominal support (80-100 cm: small, 100-140 cm: large).
- **(2B)** Shape the belt's dorsal ribs to adapt it to the patient's morphology.
- **(2C)** Adjust the height of the lumbar pad, positioning it symmetrically to the spine, to adapt it to the patient's curves. Remove the cleaning guards from the belt.
- **(2D)** Fasten one end of the belt onto one end of the abdominal support selected in (2A), covering the entire hook-and-loop disc.
- **(2E)** Position the belt so that the V-shaped ribs are on either side of the spine.
- **(2F)** Fasten the second end of the abdominal support to the belt to secure it.
- **(2G)** Adjust the abdominal support symmetrically, to obtain a tight fit without compression by centring the elasticated part under the abdomen.
- **(2H)** Adjust the elasticated straps symmetrically to get optimal support, without compressing.

CARE, MAINTENANCE, AND DISPOSAL

(illustration 3)

To preserve the belt's appearance and performance, make sure that all hook-and-loop fasteners are protected:

- **(3A)** Detach the abdominal support from the belt.
- **(3B)** Put the elasticated straps back on the outer belt disc.
- **(3C)** Put the cleaning guards on the belt's inner hook disc.
- **(3D)** Put the pad back on the inside of the belt.

Hand wash in warm soapy water (maximum 30°C) or in the machine (delicate cycle, 30°C); rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

Store the device away from sources of extreme heat (radiators, tumble dryers, etc.). The device and packaging must be disposed of in accordance with local or national environmental regulations.

SIZE CHART - TAKE MEASUREMENTS TO CHOOSE THE RIGHT ABDOMINAL SUPPORT

One-size product. For waist circumference, measure 2 cm above the iliac crest, from 80-140 cm. Choose small or large abdominal support based on this circumference:

	80 - 100 cm	100 - 140 cm
		

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GIBAUD S.A.S.U